

Bed blocks

Code	Height of bed block
MF 0/1	50mm
MF 0/2	75mm
MF 0/3	100mm
MF 0/4	150mm

Raising the height of your clients bed may enable them to get in and out of their bed more easily

- Light weight bed block with finger hole makes the blocks easy to carry.
- Durable construction means they can be reused over and over while maintaining excellent condition.
- Pyramid shape means that its self bracing & stable.
- Professionally tested to ensure your safety.
- Sealed surfaces mean cleaning is made easier.

Specifications :

Max user weight	380kg over four feet
Construction	12 mm custom wood
Cup recess	75 x 75 mm wide x 40 mm deep. Non slip rubber floor or optional caster cup floor
Finger hole	25 mm dia
Finish	Rimu stained / lacquer
Compliance	Non specified test. Based on BS EN 1022: 1997 stability of furniture


Fitting instructions

(1) Raise the bed with the assistance of a second person or a Multifit lifting jack. Place the blocks centrally under each bed leg. Lower the bed.

(2) Check that all of the bed legs are positioned safely in the blocks. Sit on the raised bed and check that it feels safe and stable.

Guarantee

- Delivery with in 5 working days or we will shout your whole OT department morning tea.
- The equipment will retain its original function for at least 2 years or we will replace it free of charge.
- Customised & developmental equipment may take longer.

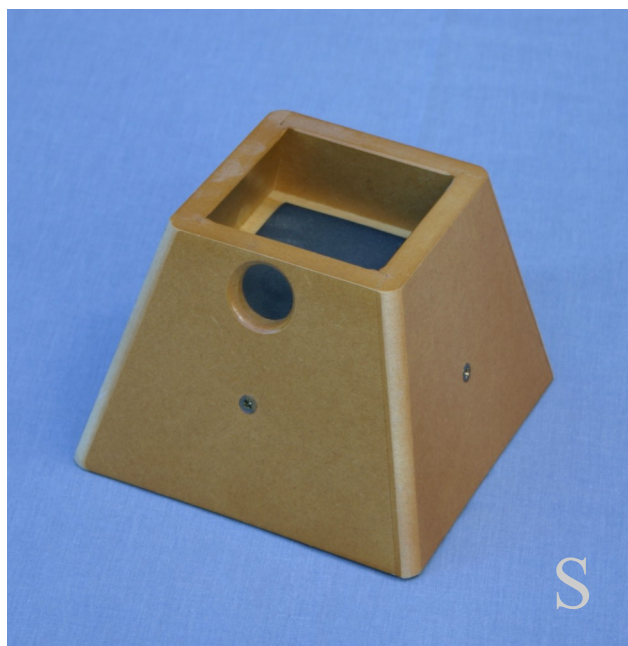


Bed block




Bed block fitted to a bed

Optional cup floor

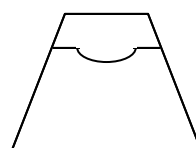

Caster cup floor

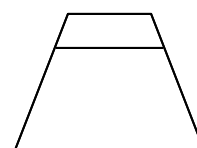

Standard flat floor

Bed blocks

Instruction sheet

Installing bed blocks



- 1/. Raise the bed with the help of an assistant or the use of a Multifit Lifting Jack.
- 2/. Place the bed blocks under each bed leg.
- 3/. Lower the bed.
- 4/. Check that all of the bed legs are positioned safely in the bed blocks.
- 5/. Sit on the raised bed and check that it feels safe and stable .



Cleaning

All surfaces are pre finished and only require dusting occasionally. Reissue clean up can be done with any mild detergent and marks can be removed with Jiff.

Parts replacement

Self adhesive rubber mat - part number MF0 - RM

All parts are available by faxing Multifit on 0800 102 800

Demonstrations and training

Available on request. If you would like a demonstration, please phone 0800 102 803

Multifit products are also available for demonstration as required at the Show your ability expo in March each year.

Safety first

This equipment is made with the best materials, workmanship, and design. As bed designs are many and varied, it is up to the person fitting the bed block, to determine its suitability for the bed you intend to raise. We have raisers made to suit most beds. If you have any doubt , please contact us for advise. We can also make a raiser up especially for your needs.

Trial equipment

If you have this equipment on trial and it does not suit your client (for what ever reason) we are only too happy to have it back.

- If unsuccessful the trial equipment must be returned in original condition within 14 days, in order to get a full credit.
- Return the equipment to : **Multifit, 10 Pohutu Street, Whakatane.**
- Please be aware that you will have to pay the return freight charges.
- Please advise us if you trial with another client.
- If the trial is successful payment will be due on the 20th of the month following

Guarantee

- Delivery : Delivery is within 5 working days, or we will shout your whole OT dept morning tea.
- Will it fit : This equipment will fit or we will customize one that will.
- How long will it last : This equipment will retain its original function for at least 2 years or we will replace it free of charge.

Reusability / cost effectiveness

We have tracked the reuse of equipment over the years. Original Multifit raisers made in 1990 are still in good service today.

Research and development

Multifit equipment is unique to our company, because our designs are developed through feed back from NZ Occupational Therapists and only sold through our business. We welcome your feed back .

Web site (www.multifit.co.nz)

Keep a look out for new products and information on our ever expanding range of unique products, as well as our news letter for updates and inspiration.

Contact details

Multifit Hospital Supplies. Ph 0800 102 803—Fax 0800 102 800 - E Mail info@mutifit.co.nz - P O Box 748 - 10 Pohutu, Street, Whakatane

Mission statement

Quality of life is precious. Our mission is :

To provide you with useful, quality, and long life equipment, that enables your clients to make the most of their lives. Serving people with disabilities since 1990