

Multifit LegLifter Mk3L

Gentle bedside assistance for the elderly and disabled



Increased independence for patients, easier management for health professionals

The Multifit LegLifter is an appliance designed to provide independence for elderly and disabled people who are unable to lift their legs from a bedside sitting position onto the bed. Depending on the level of disability, the task of getting into bed safely can be performed with little or no assistance from another person.

Easy to use

With simple push-button commands, the Multifit LegLifter gently carries the legs from the side of the bed to the prone position on the bed surface, then returns the leg-rest to its bedside position.

Smooth, silent action

Precision-engineered componentry and robust construction ensures quiet performance and safe, gentle operation.



Easy operation and safe, friendly ergonomics make the Multifit LegLifter an ideal bedside companion.

NEW DESIGN FOR MULTI-LEVEL BEDS

Adaptable for various domestic bed designs

Multifit LegLifter Mk3L is designed for installation to typical domestic slat, frame or innerspring beds. LegLifter is height-adjustable from 300mm to 700mm to fit high-level or low-level beds. Special adaptations to profiling bed designs can be supplied on request.

Installs in minutes

Multifit LegLifter installation for domestic beds and many profiling beds takes from five to ten minutes. Allen-key tools are supplied for assembly.

Compact, portable and slim

Compact dimensions and 9 Kg weight make LegLifter Mk3L easy to carry and transport, even in a small vehicle. The slim profile of the installed unit fits close beside the bed when not in operation.

High performance and durability

Multifit LegLifter Mk3L has been tested successfully for durability to 10,000 cycles.

Mk3L features include:

- New design for low-level bed frames
- Access under bed for bed-table trolley
- Options for standard or wide leg-rests to suit specific patient needs
- Synchronised lift action to suit needs of patients with diminished balancing ability
- Handset light to show when leg-rest is lowered safely to the bedside
- Refined ergonomics, slimmer profile
- Optional battery backup



Single-leg design provides space for bed-table trolley access and standing space for bed-making tasks.

Multifit Mk3L

Multifit LegLifter Mk3L



Specifying and ordering the Multifit LegLifter for individual requirements

The unique Multifit LegLifter design lends itself to many types of bed frame styles, both domestic and profiling, to suit access from either side of the bed.



The Multifit LegLifter utilises a slim, strong stabiliser which fits under the mattress to ensure sturdy installation and smooth operation. Multifit LegLifter is equipped for fitting to innerspring, frame, slat and box bed-bases. Some profiling bed frames may require custom-designed fittings to suit the type and width of bed frame. When making enquiries or ordering, specify:

- Domestic frame type and width, **or**
- Profiling bed brand or manufacturer
- Access preference from left or right side of bed, viewed from the typical prone position in the bed:

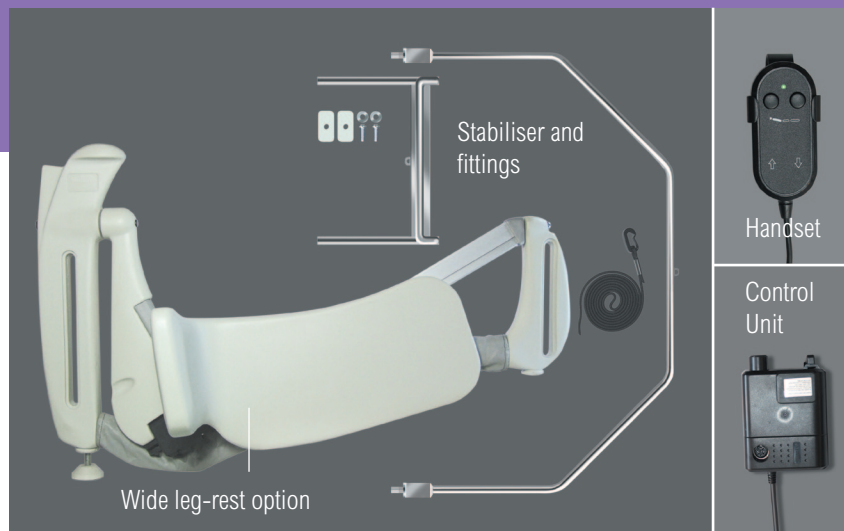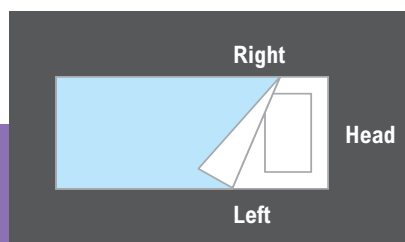

| | | |
|---|--|--------------|
| LegLifter unit dimensions | Width 1050mm | Height 660mm |
| Height adjustment range | 300mm to 700mm | |
| Distance from bedframe to platform of folded leg-rest | 60mm | |
| Total weight including packaging for transit | 16 Kg | |
| Weight of LegLifter unit | 9 Kg | |
| Installation time for standard domestic bed | From 5 to 10 minutes | |
| Maximum leg-load weight | Tested to 67 Kg (legs must fit within legrest) | |

■ Multifit LegLifter Mk3L is available with the option of a wider, interchangeable leg-rest to suit patients with swollen or heavier legs. Specify either standard or wide leg-rest when ordering:



Manufactured by:
 Multifit Hospital Supplies Ltd
 PO Box 748 Whakatane 3158, New Zealand



© 2008-2010 Multifit Hospital Supplies Ltd. Published June 2010

Low maintenance, easy care

Innovative design and sturdy engineering offer easy cleaning and low maintenance.

Safety features

- Low 28V power to the LegLifter
- Automatic sensor turns motor off in the event of obstruction behind the footrest
- Telescopic support prevents leg-rest lowering under power against obstruction
- Rounded edges prevent injury
- Handset light turns off when leg-rest is fully and safely lowered beside the bed.

See the LegLifter Mk3L in action!

To view a brief video and to download more information, visit www.multifit.co.nz

Multifit Mk3L

www.multifit.co.nz
export@multifit.co.nz